


SmartPee® Dip

» Ready for a quick dip?

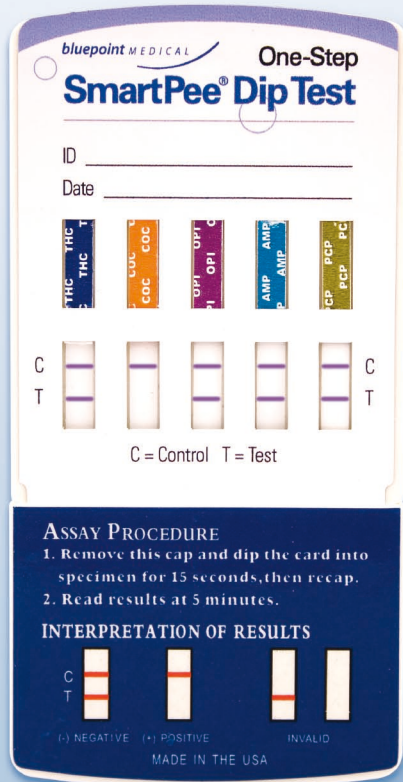

AMP AMP AMP AMP	Amphetamine
BAR BAR BAR BAR	Barbiturate
BZD BZD BZD BZD	Benzodiazepine
COC COC COC COC	Cocaine
MET MET MET MET	Methamphetamine
MTD MTD MTD MTD	Methadone
OPI OPI OPI OPI	Opiates
OXY OXY OXY OXY	Oxycodone
TCA TCA TCA TCA	Tricyclic Antidepressants
PCP PCP PCP PCP	Phencyclidine (PCP)
THC THC THC THC	Marijuana
MDMA MDMA MDMA MDMA	Ecstasy
PPX PPX PPX PPX	Propoxyphene
BUP BUP BUP BUP	Buprenorphine

» PRODUCTS that care for you «

FROM THE MAKER OF THE ONE-STEP SMARTPEE® CUP

With the One-Step SmartPee® Dip Test, you can now experience the same reliability and simplicity you've come to trust from bluepoint® Medical. We know dependability is your top priority. That's why we created an alternative test in a dip card format with the same accuracy and ease of use as the One-Step SmartPee® Test Cup.

EASY, RELIABLE, FAST

The One-Step SmartPee® Dip Test is a rapid, qualitative immunoassay for the detection of up to 10 drugs, at the SAMHSA cutoff levels, in a convenient format. Simply insert the test strips in the collected urine sample for 10-15 seconds and read the results at 5 minutes.

- Accurate results in 5 minutes
- Easy to read
- Simple procedure
- 18-month shelf life
- No instrumentation
- Available in single- and multiple-test panels
- No urine to pipette

Preparation, Specimen Collection, Test Procedure and Interpretation of Results



Collectors should be trained on the procedures for operating the tests. Do not break the seal of the protective pouch until ready to begin testing. Visually inspect the foil package to ensure it has not been compromised before beginning the test. If the package is not intact, the integrity of the test may be compromised.

1. Fresh urine specimens should be collected directly into the collection cup and do not require any special handling or pre-treatment.
2. The SmartPee® Dip Test should be used with a collection cup that employs a temperature strip to validate that the urine is a freshly voided specimen. This device should be checked immediately after collection.
3. Test procedure: Tear open and remove the Dip Test. Tear open the foil pouch and remove the Dip Test.
4. Issue the collection cup to the individual to be tested. Instruct the donor not to flush the toilet or turn the water on and to open the door when they are done.
5. Have the individual urinate directly into the collection cup. Ensure that the specimen contains at least 30 ml of urine.
6. Check temperature (normal range between 90°F - 100 °F or 32 °C - 38 °C) and observe specimen for foreign material and/or discoloration.
7. Lower the Dip Test into the specimen, ensuring that the bottom of the test strips contact the urine. Plastic dip device should not touch the specimen (urine).
8. Replace the cap covering the ends of the test strip.
9. Read results at 5 minutes.

Drug test results

PRESUMPTIVE POSITIVE

One colored line only is visible in each Control Zone. No colored line whatsoever appears in the appropriate Test Zone, indicating a presumptive positive result for the corresponding drug of that specific Test Zone. Send this urine specimen to a certified laboratory for confirmation.

NEGATIVE

Two colored lines are visible in each Control Zone and in the appropriate Test Zone, indicating that the concentration of the drug is below the detection limit of the test or the drug is not present.

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
RETEST

No lines: If a colored line is not visible in the Control Zone, the test is invalid. Another test should be run to re-evaluate the specimen. Each strip in the SmartPee® Dip Test is read and functions independently. An invalid result on one test strip does not invalidate other results derived from the same device.

Note: There is no meaning attributed to line color intensity or width. Any evidence of a line should be considered a line. In order to prevent any incorrect results, the test results should not be interpreted after 10 minutes.

If confirmation is required, secure specimen in appropriate shipping container and send to Confirmation Lab.

bluepoint® MEDICAL sales partner



SmartPee® is a registered trademark of bluepoint® MEDICAL.

For further information, please visit www.bluepoint-medical.com



Dip device into collected urine for 15 seconds



Replace cap



Read results at 5 minutes

